

Innovative Endo Products



5, Simcha Holzberg St. Kiryat-Ono 55022, Israel
Tel: +972-3-7360099, Fax: +972-3-7360097
export@romidan.com | www.romidan.com

Distributed By:



Romidan Innovative Endo Products

Autoclavable Endo Files Organizer
User Instructions and Sterilization Procedures



www.romidan.com

Dear Dental Professional.

Your new 'Endo-Files Organizer' enables you to sterilize -in an Autoclave-, all types of files, including Manuals and NiTi files.

To maximize all benefits of your new Endo-Files Organizer, we strongly recommend to follow all Instructions, precisely as specified below:

Sterilization Instructions:

Before placing the Endo-Files Organizer into the Autoclave:

- Remove the Upper Cover of the Endo-Files Organizer from the Lower Container part. see Pic. **A**
- Remove all Top Covers from all 6 (six) Cylinder Containers see Pic. **A**
- Place all Cylinder Container Top Covers inside the Upper Cover. see Pic. **A**

Placing the 'Endo-Files Organizer' into an Autoclave:

- Place the Upper Cover and the Lower Container on a sterilization sleeve or a paper towel, to prevent direct contact between the metal tray and the various plastic parts. see Pic. **A**



Sterilization conditions in an Autoclave:

- Maximum temperature: 134°C
- Sterilizing time: Up-to 5 minutes
- Drying time: 10-15 minutes in an Autoclave

Caution:

Temperatures higher than 134°C may damage the 'Endo-Files Organizer' or the files.

After Sterilization is completed:

- Hold the Upper Cover - which contains the six Cylinder Container Top Covers -, in your one hand. see Pic. **B**
- Hold the Lower Container - which contains the six Cylinder Containers, in your other hand. see Pic. **B**
- Position both the Upper Cover and the Lower Container facing each other at an angle. see Pic. **B**
- Securely press to close both parts together. see Pic. **C**

Your Endo-Files Organizer and files are now sterilized and ready for use.

To use your Endo-Files Organizer :

- Remove the Upper Cover from Lower Container. see Pic. **D**
The inner Cylinder Containers remain closed and sterile.
- Pull out selected Cylinder Containers, containing sterile endodontic files-, required for your root canal treatment according to your preference. see Pic. **E**

To guarantee your full satisfaction from your 'Endo-Files Organizer' you must follow these directions precisely.